




**MILESTONE**  
H E L P I N G  
P A T I E N T S

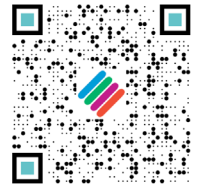

# HistoDream TS

## Biospecimen Tracking System

Touchscreen tablet with barcode reader for case traceability




# MILESTONE'S HISTODREAM TS - TRACKING SYSTEM



**The HistoDream Tracking System** empowers Histotechs to visually match grossed specimens to processed specimens before embedding by retrieving the digitized images of each cassette taken at the time of grossing and comparing those images to the cassette at the time of embedding. Detection of mismatched number, size or type of specimens before embedding allows for early corrective actions. Available for any type of embedding station.

## TISSUE PROCESSING

### WHEN GROSSING

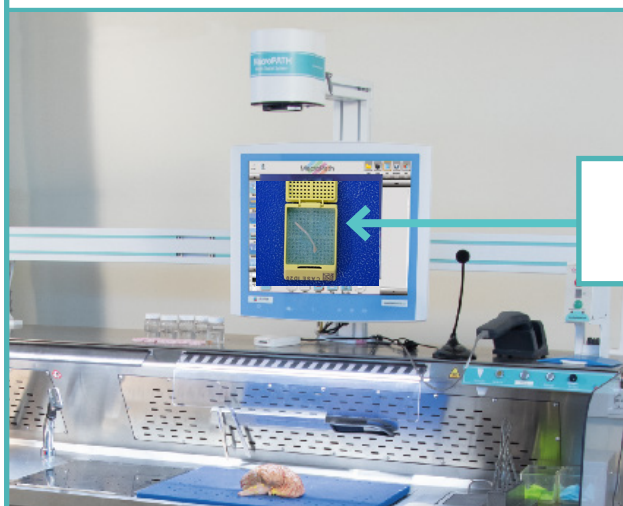


### MILESTONE'S MACROPATH

#### 3-step Process in Less than 15 Seconds:

1. Start a new case using MacroPATH Viewer Software by scanning the cassette barcode
2. Position your tissues inside the cassette
3. Take a photo with MacroPATH to document the number of tissues, size and tissue type

### Visual Matching

### WHEN EMBEDDING

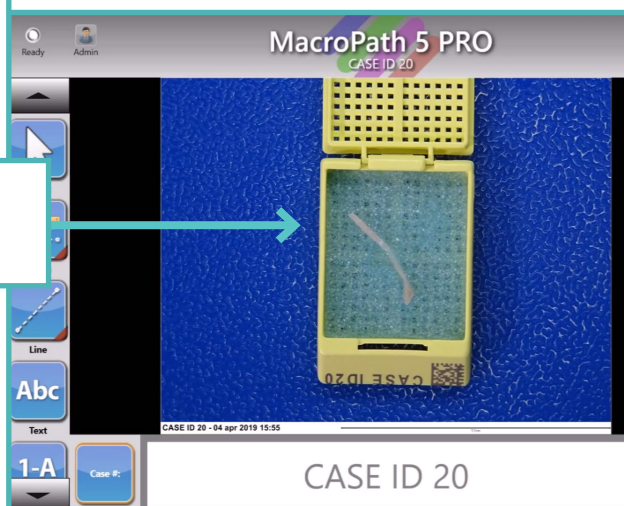

### MILESTONE'S HISTODREAM TS

#### Validate Cassette and Tissue Integrity:

1. Scan cassette barcode. HistoDream TS will retrieve the image of the cassette and tissue(s), saved at the time of grossing
2. Validate number of specimens, type and size
3. Act per internal procedure in case of mismatch

Requirements: MacroPATH Digital Gross Imaging System in your grossing room and LAN network port in the embedding room



## US Headquarters

MILESTONE MEDICAL TECHNOLOGIES, INC.

6475 Technology Avenue, Suite F - Kalamazoo, MI 49009

Tel: 269-488-4950 - Toll-free: 866-995-5300 - Fax: 269-488-4949

email: [info@milestonemed.com](mailto:info@milestonemed.com) - [www.milestonemed.com](http://www.milestonemed.com)