



MILESTONE
H E L P I N G
P A T I E N T S



T-Tracker

**Automatic & Reliable Tracking
for Enhanced Patient Safety**



Three-in-One
Scan, Verify, Track

T-Tracker

T-Tracker replaces the tedious, time-consuming, and error-prone task of manual cassette scanning with automated, error-free, three-in-one technology for cassette scanning, verifying, and tracking. T-Tracker is the sole tracking solution in today's market that combines two advanced technologies to ensure reliable tracking, catch potential errors, and secure true patient safety. T-Tracker's dual sensor technology combines a laser sensor and optical sensor, which work together with an enhanced software algorithm. Together, they count cassettes, scan barcodes, and check for errors, such as duplicate, missing, or unreadable cassettes/codes, to ensure accurate, precise, and verifiable tracking of all cassettes.

DUAL SENSOR TECHNOLOGY
OPTICAL AND LASER SENSORS
for true error-free tracking

BUILT-IN EXHAUST FAN
for reducing formalin exposure

EASY ACCESS POWER BUTTON
with light indicator for visibility


SMART USER INTERFACE
for one-click tracking (optional)

EMERGENCY STOP BUTTON
for additional operator safety


INTERCHANGEABLE TRAY
for use with multiple rack types

SLIDE-OUT RACK DRAWER
with self-centering mechanism

TRANSPARENT SAFETY SHIELD
for reducing formalin exposure

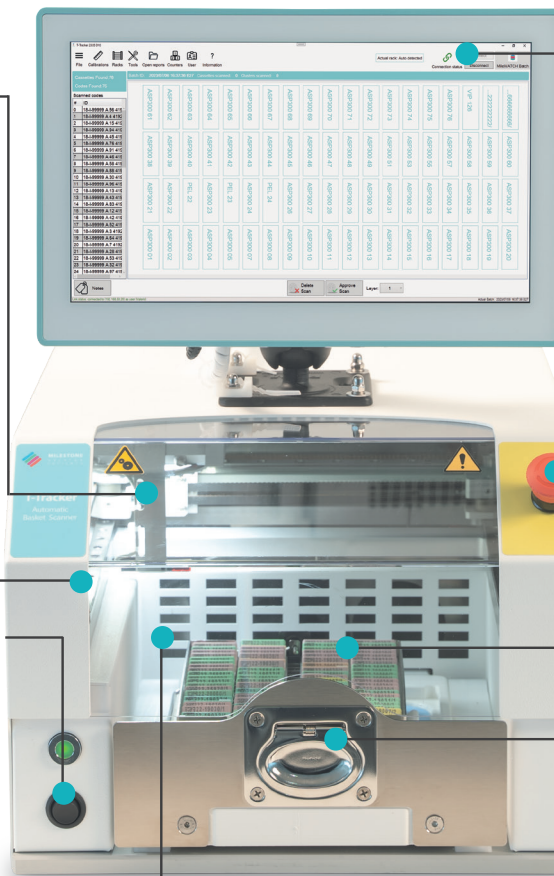

SEE T-TRACKER IN ACTION

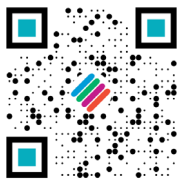

FULL TRACEABILITY AND COMPLETE DOCUMENTATION

DUAL SENSOR TECHNOLOGY

With just a single click, the dual sensor technology of T-Tracker scans, verifies, and tracks the loaded cassettes, getting it right the first time. The laser and optical sensors work in perfect synergy to deliver the results you need, fast.




LASER SENSOR
Checks the presence and counts every cassette in the basket



OPTICAL SENSOR
Scans the basket and detects, reads, and collects the cassette codes

Tracking Overview

Summary of the scan, including: number of cassettes found and codes found. The system will provide warning messages here if mismatched and/or duplicate codes are found.

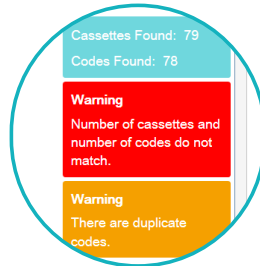

Basket Graphic Representation



Visual representation of scanned cassettes:

- With respective code (blue)
- With missing and/or unreadable code (red)
- With duplicate codes (orange)



Scanned Codes and Notes

List of scanned codes present in the basket. The notes function enables the addition of comments to each cassette.

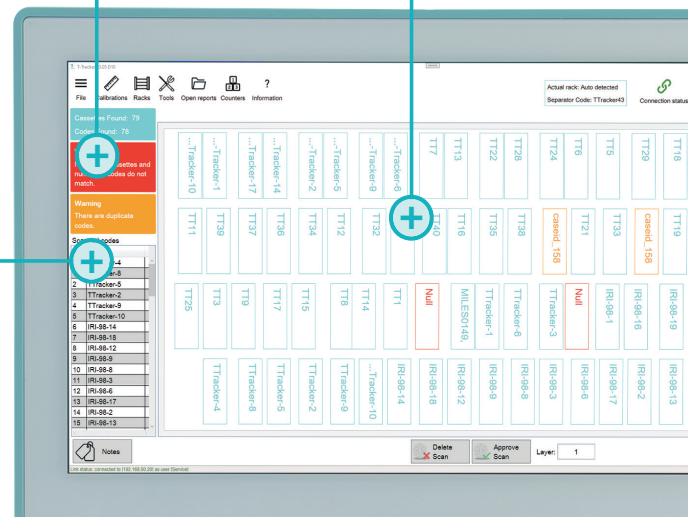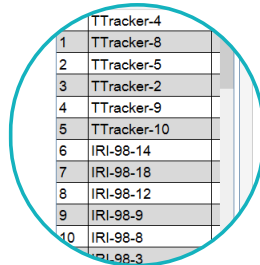

KEY BENEFITS

TIME SAVINGS

- Eliminate manual scanning
- Up to 97% time savings!

ERROR-FREE AUTOMATION

- Dual sensor technology
- Accurate count and scan of all cassettes

SAFETY

- Built-in exhaust fan & safety shield
- Formalin absorbing pads (optional)

CHAIN OF CUSTODY

- Real time batch tracking
- Precise and accurate tracking of each case

COST SAVINGS

- No maintenance or running costs
- Save on tech time utilizing automation

CONNECTIVITY & COMPATIBILITY

- Integrate with MileWATCH or LIS
- Compatible with most rack types


PRODUCT FEATURES

Available in two configurations:

RACK COMPATIBILITY: Rectangular and Circular



T-Tracker
With built-in PC and monitor

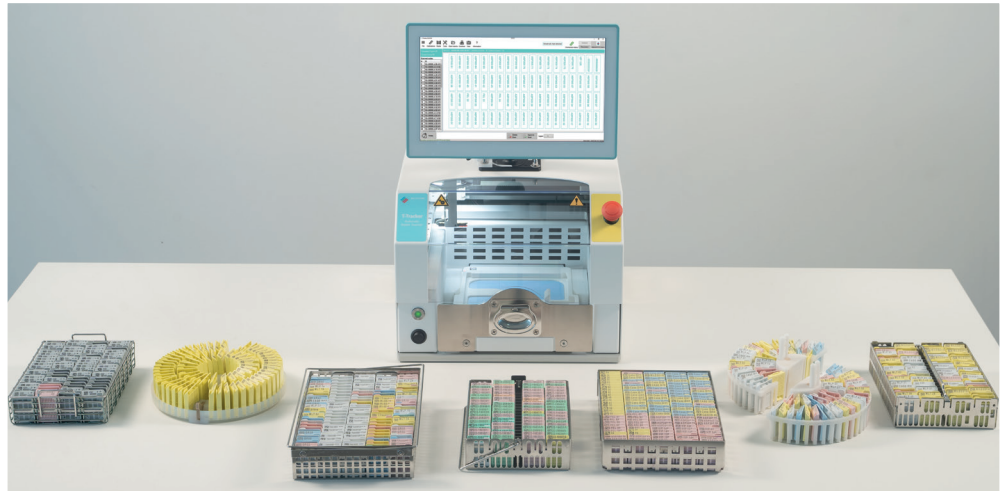


T-Tracker
Without PC and monitor

T-Tracker offers wide compatibility with different types of racks and starting the process is simple. Place the rack into the self-centering tray, slide it in, and press 'START' on the terminal. **The system automatically recognizes the rack type** to secure precise tracking. Moreover, the use of spacers simplifies the positioning of the cassettes, ensuring a precise reading and stabilizing the cassette even during transportation. This approach also offers a homogeneous flow of reagents between the specimens during processing.

RELATED PRODUCTS



R-Tracker:
Automatic Rack
Scanner

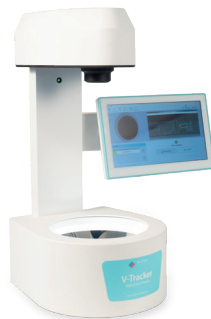

V-Tracker
Automatic Vial
Scanner



MileWATCH:
24/7 Monitoring
& Tracking System



MAGNUS:
Rapid, Automatic
Tissue Processor



US Headquarters

MILESTONE MEDICAL TECHNOLOGIES, INC.

6475 Technology Avenue, Suite F - Kalamazoo, MI 49009

Tel: 269-488-4950 - Toll-free: 866-995-5300 - Fax: 269-488-4949

email: info@milestonemed.com - www.milestonemed.com