



HISTOSMATE

An innovative, safe, rapid solution for the efficient cleaning of wax contaminated racks, molds and forceps.

✓ **FULLY AUTOMATIC OPERATION**

- 1) Load the rack - The cavity holds non-toxic isoparaffin at a constant 50°C.
- 2) Push START - The heated isoparaffin is constantly stirred to assure rapid and efficient elimination of wax traces.
- 3) Unload the clean rack - **The entire process takes 7 minutes.**

✓ **NO XYLENE**



PATHOS RACK AFTER PROCESSING

**AUTOMATIC,
RAPID RACK
CLEANING**



AFTER 7' CLEANING WITH HISTOMATE



Need more wax impregnation time?

Dual use: the HistosMATE as a stand alone, automatic wax impregnation module

- 1) Simply load rack in the cavity which holds paraffin wax at a user's selected temperature.
- 2) Set desired impregnation time (up to 24 hours).
- 3) The optimum combination of powerful magnetic stirring and homogeneous temperature distribution assures a complete wax impregnation of specimens.

HistosMATE
universal stainless
steel holder suitable
for all types of racks




OPTION

HistosMATE steel basket to automatically remove wax traces from metal molds and forceps.



Before

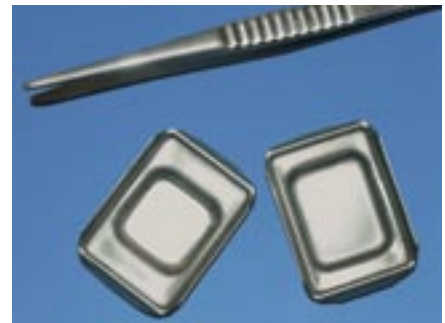

Seven minutes later



MILESTONE

UNI EN ISO 9001 : 2000 CERTIFIED

MILESTONE S.r.l.
Via Fatebenefratelli 1/5
24010 Sorisole (BG) Italy
Tel. +39-035-4128264
Tel. +39-035-572735
Fax +39-035-575498
e-mail: medical@milestonesrl.com
customersupport@milestonemedsrl.com
www.milestonemedsrl.com

MILESTONE MEDICAL
25 Controls Drive
Shelton, CT 06484 - USA
Tel: (203) 925-4240
Toll-free: (866) 995-5300
Fax: (203) 925-4241
e-mail: info@milestonemed.com
www.milestonemed.com